

Before you start

Profilometry has become the gold standard for scleral lens fitting. The **Eye Surface Profiler** is an easy to learn and easy to use device that is the world leader in ocular Profilometry. We have enjoyed working closely with our lab partners for many years with the aim of providing the best outcome for our patients. And now we are thrilled to introduce **DirectConnect™**.

DirectConnect with **ScanFitPRO** combines the expertise of the two companies to deliver a scleral lens designed to match the exact scleral contours of a patient's individual eye and a faster ordering process.

This guide explains how to use the DirectConnect feature step-by-step. For any feedback or questions please contact us at support@eaglet-eye.com.

1

ScanFitPRO Software

To be able to use the DirectConnect functionality for your ScanFitPRO ordering, you first need to purchase and install the ScanFitPRO software in your computer. Contact EyePrint PROSTHETICS

- 1) via website: <https://eyeprintpro.com/contact>; check the ScanFitPRO box, or
- 2) via email: naomi@eyeprintpro.com.


2

Select lens design

- 1) Go to **FIRST FIT LENS** on the left menu.
- 2) Select **Supplier: ScanFitPRO**.

NOTE: ScanFitPRO will appear in the Supplier drop-down list only if its SW is installed in your computer.

- 3) Click on **DirectConnect**.

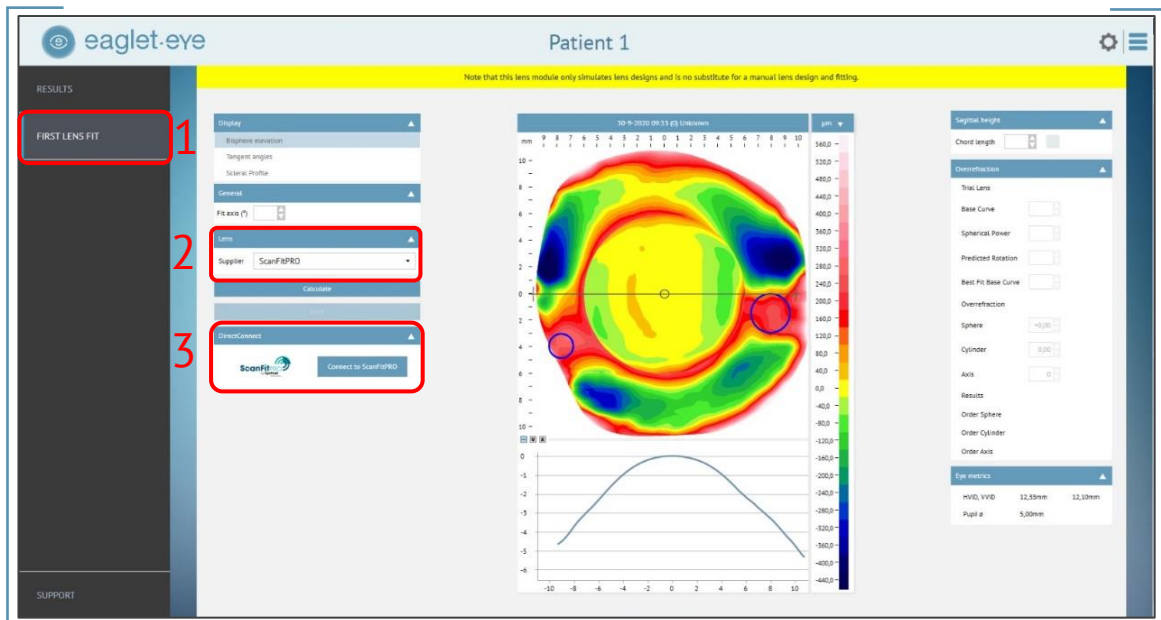

3

Order the lens directly

The **ScanFitPRO software** will open on the screen where you can then continue further customizing the ScanFitPRO lens.

For more details, please contact the Eaglet team (support@eaglet-eye.com) or your ScanFitPRO representative.